



THE NEUROSIGN[®] V4

INTRAOPERATIVE NERVE MONITOR

PRESERVING NERVE INTEGRITY



Reference Guide

NOI9-EN-04
January 2021

CE
2797





Wipe clean only. Do not soak.

This reference guide should consist of 6 double sided sheets.

Indications, contraindications, warnings, precautions and other safety information are contained in the Neurosign® V4 User Manual NOP08.

Consult the Neurosign® V4 User Manual before use.

Discover more at;
neurosignsurgical.com





Important: Before use, make sure you have read and understood the Neurosign® V4 Intraoperative Nerve Monitor User Manual (NOP08). Copies can be obtained from your local Neurosign representative.

PRE-USE CHECKS


- Before use, the equipment and accessories should be checked for any damage or tampering. See “Damage Inspection” in the user manual (NOP08) for information.

For Cleaning and Disinfection information refer to the user manual (NOP08).


CONNECTION




FRONT UI VIEW

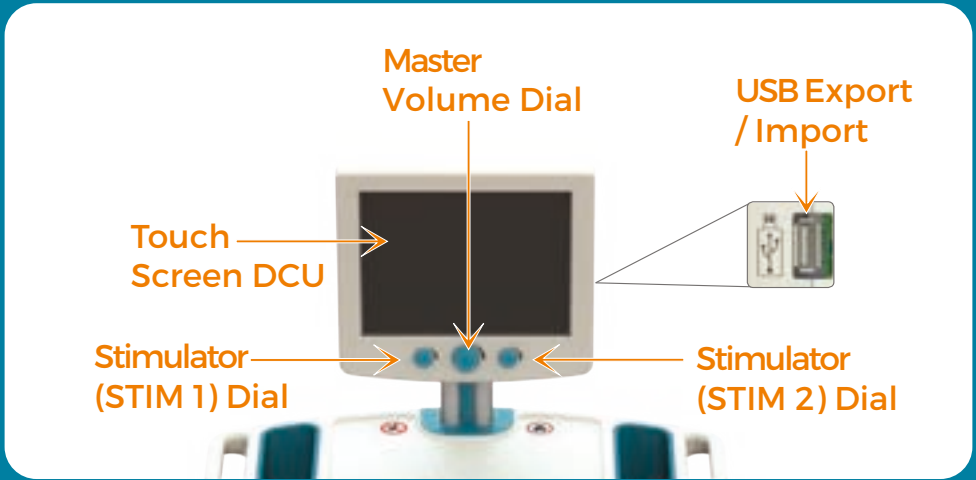

REAR UI VIEW


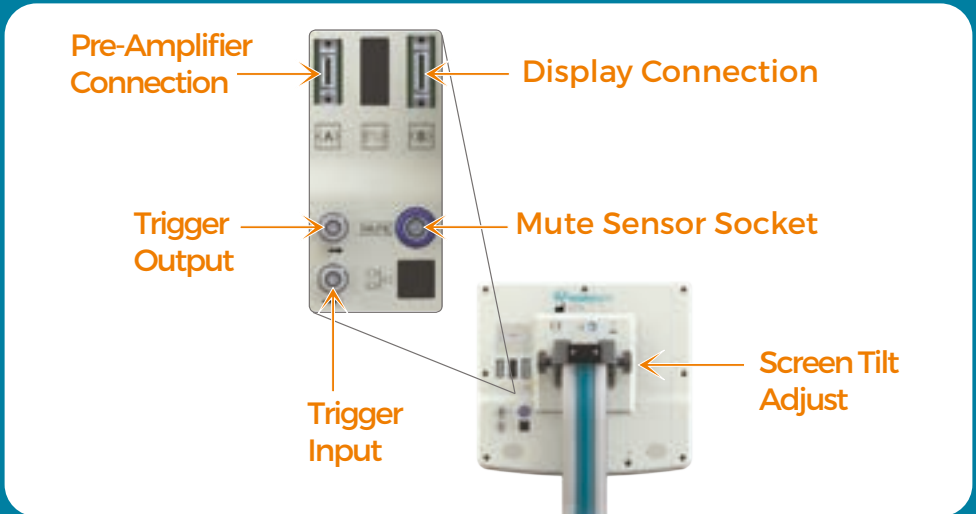


PRE-AMPLIFIER

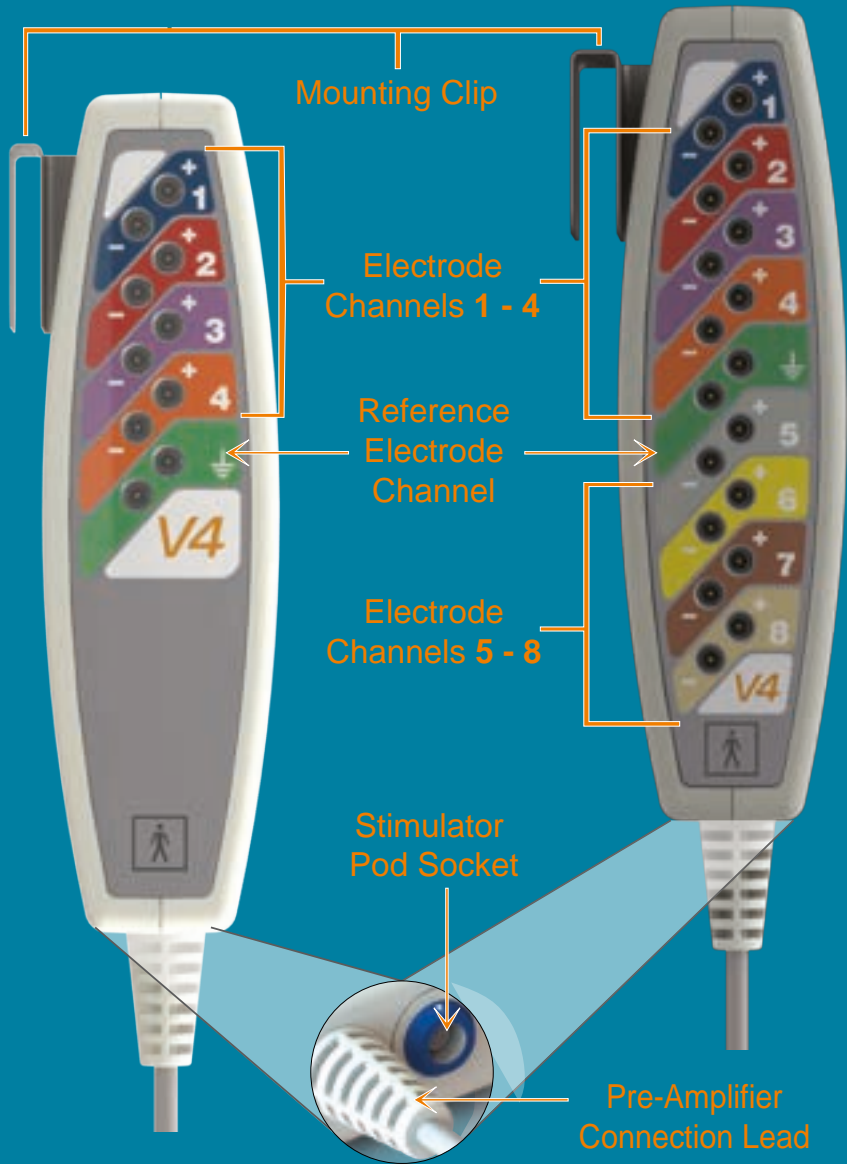

MUTE SENSOR



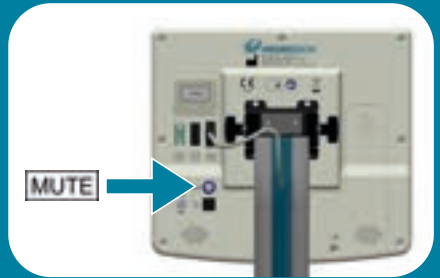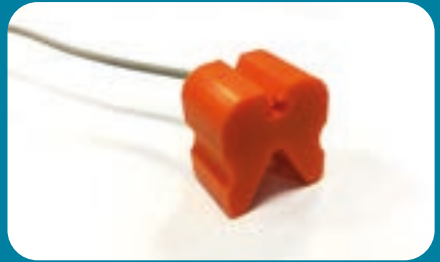

- If using, attach the mute sensor to the active lead of the electrocautery/diathermy device
- Plug the mute sensor cable into the socket on the rear of the touch screen DCU **MUTE** (right).
- **Do not** connect the mute sensor to the nerve monitor without attaching the clamp to the electrocautery cable or spurious muting of the nerve monitor may occur.





STIMULATOR POD



QUICK START SESSION (DEFAULT)

- On the **Home screen**, select 
- Select **Default** and press 
- Select a surgery procedure and press 
- Press  to check the electrode impedance.

All indicators should show green . If not, check/reposition the electrodes (refer to “impedance check” in the user manual for more information).

- Drape the patient
- Press  to begin monitoring
- Press  to end monitoring

Note: In default mode, procedure settings can be altered via the monitoring menu but the settings cannot be saved.

Patient Reports

Patient reports are not available in the default profile. In order to save patient reports you must create a surgeon profile (see advanced features). A surgeon profile will also allow you to save your custom procedure settings.

IN-SURGERY MENU

For more information refer to the user manual (NOP08).



Home



System Settings



Procedure Settings



Screen Capture (Not available in default mode)



Show/Hide Latency Cursors



Procedure Report (Not available in default mode)



Waveform Settings



Impedance Check




Start




Stop

ACTIVE MONITORING SCREEN

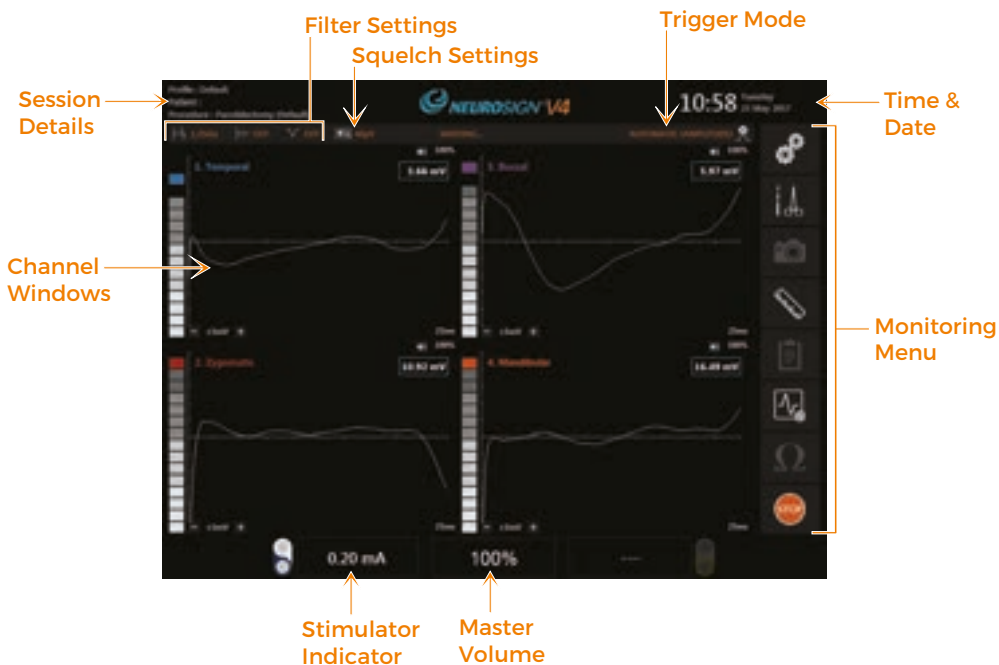


CHANNEL WINDOW

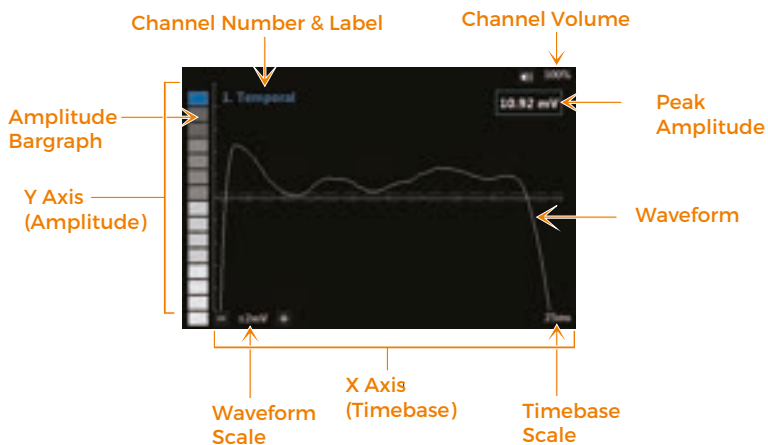

STIMULATOR ADJUST

- The **stimulator current** is displayed at the bottom of the 'active monitoring screen'. Set the stimulator current using the left stimulator dial.
- When the Stimulator Probe is used, the **stimulator indicator** should change to green (as below):

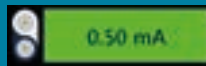

- If the stimulator indicator does not change colour, the probe has not stimulated at the set current (refer to "Troubleshooting" in the User Manual for more information).

VOLUME CONTROLS

- The **master volume** is shown in the middle at the bottom of the 'active monitoring screen'. Set the master volume using the central master volume dial.
- To **mute** all channels temporarily, press and hold the master volume dial for 2 seconds. The master volume display will then show the following:



- The **individual channel volume** is shown in the top right of each channel window. You can change the individual channel volume by pressing it.

ALARMS

- In the event of an alarm (below) follow the on-screen instructions.



- More information can be found in the “Troubleshooting” section of the user manual.


MAJOR FAULT

- In the event of a major fault (below) discontinue use and refer to the user manual.






ADVANCED FEATURES

Custom surgeon profiles

To save patient reports and store your settings, you must create a password protected surgeon profile by pressing .

Refer to “Profile Management” in the user manual.

Custom procedures

You can save customised procedure settings for future use. Press  to duplicate a default procedure and then press  to edit it. Once you have finished making changes press  to continue (not available in default mode).

Refer to “Custom Procedure” in the user manual.

Patient reports

Note: Not available in default mode. To save and access patient reports, patient details **must** be entered when prompted.

From the home screen press  to access the **Report Management Screen**. Press  to view the selected report.

Refer to “Patient Reports” in the user manual.

Settings

From the **Home Screen**, press  to access the Settings menu.

For more information see “Settings” in the user manual.