Technomed
medical accessories

Technomed Europe
Wiebachstraat 25a
6466 NG Kerkrade
The Netherlands

CH

REP

Switzerland

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

United Kingdom

The Magstim Company Ltd.
Spring Gardens,
Whitland, SA34 0HR,
United Kingdom

Australia

ACRA Regulatory Services Pty Ltd
7/84 Poinciana Avenue,
Tewantin, QLD 4565
Australia

REUSABLE EEG CUP AND EAR CLIP ELECTRODES**Intended use**

Reusable EEG cup and ear clip electrodes are intended for non-invasive use with DIN 42802 compatible recording and monitoring equipment in electroencephalography, electromyography, and evoked potential studies. Electrode preparation and application should be carried out by a qualified healthcare professional and electrodes can be used on the general population at the discretion of a healthcare professional.

Note: Reusable EEG cup and ear clip electrodes are intended to be used only on intact skin. Do not use these products when suspecting any skin infection to avoid cross contamination. Do not use these products for any kind of stimulation, as it may lead to skin damage.

Use of product

- Clean and disinfect patient's skin prior to application of reusable EEG cup and ear clip electrodes according to procedures of your healthcare facility.
- The electrodes should be applied with the use of a conductive EEG gel/paste with or without collodion. The conductive gel may be applied by a syringe with a blunt needle through the hole on top of the electrode.

Cleaning

Electrodes should be thoroughly cleaned after each use and prior to disinfection. Immediately after use, wipe cup and cable using a soft cloth dipped in lukewarm water (at temperatures below 43°C (110°F)) with a mild detergent/enzymatic product to remove foreign material, including electrode paste, gel, or adhesives. If necessary, use a soft, clean brush to apply enough gentle friction to remove all foreign materials. Excessive pressure could bend the cup, break the cup from the spring or from the connection to the cable, stretch the cable jacket and break the internal wires, damaging the electrode. Continued flexing of cable in use and cleaning can also break the internal wires. After wiping clean, rinse cup and cable with lukewarm water to prevent coagulation and thus assisting in the removal of protein substances. Carefully inspect the electrode to ensure the complete removal of any contamination. Do not immerse or rinse the touch-proof EEG DIN connector in any solution, as it may damage the product.

Note: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Prolonged immersion (more than 1 hour) in alcohol or mild solvents, detergent solutions or highly alkaline solutions may affect the cable and electrode jacket flexibility.

INSTRUCTIONS FOR USE

REUSABLE EEG CUP AND EAR CLIP ELECTRODES

Disinfection

After cleaning, electrodes may be disinfected with the use of 70% ethanol. The cup, ear clip and the cable can be briefly (5-15 minutes) immersed in a 70% ethanol or wiped with a soft cloth dipped in a disinfectant solution. The touch-proof EEG DIN connector should never be immersed in any liquid.

Drying

After cleaning and disinfection, dry the electrode with a clean, soft, low-linting cloth to remove any water or disinfection solution.

Sterilization

The reusable EEG cup and ear clip electrodes are not designed to sustain any type of sterilization, as it may damage the product.

Inspection

Visually inspect the electrode for deterioration or excessive wear prior to each use. If cable, connector, spring, or cup have any discolouration, fracture, damaged wire insulation, or unnatural angle, discard the electrode according to your hospital's procedure for medical waste.

Storage and Transport

Protect the electrode from damage and contamination during storage and transport. When not in use, electrode cables should be loosely coiled and stored at room temperature. Do not wrap electrode cables around equipment casing to avoid damaging internal wires. When transporting the products outside its main area of use, pack them in a resealable plastic bag to prevent damage or contamination.

Note: All serious incidents associated with the use of reusable EEG cup and ear clip electrodes should be reported to Technomed Europe at qualitytechnomed@welcony.com and to a competent authority of the country where user or patient is established.

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