DASE

DISPOSABLE ADHESIVE SURFACE ELECTRODES (single/twisted/parallel)

- 4-disk
- Ground
- NCV kit

EN: INSTRUCTIONS FOR USE

BG:	ИНСТРУКЦИИ ЗА УПОТРЕБА
CS:	NÁVOD K POUŽITÍ
DA:	BRUGSANVISNING
DE:	GEBRAUCHSANWEISUNG
EL:	ΟΔΗΓΙΕΣ ΧΡΗΣΗΣ
ES:	INSTRUCCIONES DE USO
ET:	KASUTUSJUHEND
FI:	KÄYTTÖOHJEET
FR:	CONSIGNES D'UTILISATION
HR:	UPUTE ZA UPORABU
HU:	HASZNÁLATI ÚTMUTATÓ
IS:	NOTKUNARLEIDBEININGAR
IT:	ISTRUZIONI PER L'USO
JA:	使用説明書
KO:	사용 설명서
LT:	NAUDOJIMO INSTRUKCIJOS
LV:	NORĀDĪJUMI PAR LIETOŠANU
MK:	УПАТСТВО ЗА УПОТРЕБА
NL:	GEbruiksaanwijzing
NO:	BRUGSANVISNING
PL:	INSTRUKCJA STOSOWANIA
PT:	INSTRUÇÕES DE UTILIZAÇÃO
RO:	INSTRUCȚIUNI DE UTILIZARE
RU:	ИНСТРУКЦИЯ ПО ПРИМЕНЕНИЮ
SK:	POKYN Y NA POUŽITIE
SL:	NAVODILA ZA UPORABO
SR:	UPUTSTVO ZA UPOTREBU
SV:	BRUGSANVISNING
TR:	KULLANIM TALİMATLARI

INSTRUCTIONS FOR USE

DISPOSABLE ADHESIVE SURFACE ELECTRODES

single/twisted/parallel: 4-disk, Ground, NCV kit

EN

Intended use and indications for use

Disposable Adhesive Surface Electrodes are non-sterile medical devices intended for use with recording / monitoring and stimulating equipment for the purpose of stimulating / recording of biopotential signals including electroencephalography (EEG), surface electromyography (EMG), nerve conduction and evoked potential (EP) signals.

Intended users

The intended users are healthcare professionals specifically trained and certified in electrophysiology techniques.

Patient target groups

The patient target groups are any patients identified by these medical experts to benefit from the diagnostic procedures as described above.

Use of product

- * Apply only to unbroken skin. Thoroughly cleanse application site to remove lotion, dirt, etc. Dry site of application completely. Clip hair if necessary. **Do not shave or abrade area!**
- * Remove the electrodes from the package. Remove electrodes from the liner by lifting the edge of the electrode.
- * Apply electrodes to the treatment site by firmly pressing down the center of the pad first and then smoothing down to the electrode edges.
- * When removing the electrodes, turn off the recording / monitoring equipment, disconnect the lead wire from the equipment, remove the electrode from the skin, and remove possible gel residue on the skin with water and normal soap.
- * Do not remove electrodes from patients by pulling on wires.

Cautions

- * Federal law (USA) restricts this device to sale by or on the order of a physician.
- * Do not use the device if the use-before-date has passed.
- * Electrodes in an opened pouch may dry out, therefore do not open pouch until necessary.
- * For single-patient use only. Do not reuse. Cleaning and reusing the device can affect its safety, performance and effectiveness and expose patients and users to unnecessary risks such as infection and transmissible disease.
- * Do not use the device in strong electrical fields, as any induced electrical field may influence the stimulation output on the connected equipment (if applicable), may make the readings on the connected equipment unreliable and may result in localized tissue heating.
- * If irritation develops, discontinue use and consult your clinician.
- * Self-adhesive electrodes should be replaced if they no longer stick firmly to the skin.
- * The size, shape, type and electrical performance characteristics of electrodes may affect the safety and effectiveness of the electrical stimulation and recording.
- * Using stimulation electrodes that are too small or incorrectly applied could result in discomfort or skin burns.

MRI safety information

Do not use in an MRI scanning environment. The device has not been evaluated for safety in an MR environment. It has not been tested for heating or unwanted movement in an MR environment. Its safety in the MR environment is unknown. Performing an MR exam on a person who has this medical device inserted or positioned on them may result in injury and / or device malfunction.

Storage, transport and handling

Store between 15°C (59°F) and 35°C (95°F).

Compatibility with accompanying devices

This device is equipped with DIN 42802 connectors and is compatible with any electromyography, intraoperative neuromonitoring and stimulating equipment that is equipped with DIN 42802 connections.

Disposal

Always discard used devices in a properly marked medical biohazard container.

Reporting incidents

All serious incidents associated with the use of this device should be reported to Technomed Europe at qualitytechnomed@welcony.com and to a competent authority of the country where the user is established.

Electronic instructions for use

Access the most recent electronic instructions for use through the link which is provided on the label of the device.

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Pty Ltd
The Magstim Company Ltd.