Technomed
medical accessories

Technomed Europe
Wiebachstraat 25a
6466 NG Kerkrade
The Netherlands

CH	REP
----	-----

Switzerland

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
Switzerland

United Kingdom

The Magstim Company Ltd.
Spring Gardens,
Whitland, SA34 0HR,
United Kingdom

Australia

ACRA Regulatory Services Pty Ltd
7/84 Poinciana Avenue,
Tewantin, QLD 4565
Australia

INSTRUCTIONS FOR USE

REUSABLE EMG ELECTRODE CABLES

Intended use

Reusable EMG electrode cables are intended to connect an EMG needle electrode to a recording/monitoring device for the recording and monitoring of biopotential signals, such as electromyography (EMG) and nerve potential signals. Reusable EMG electrode cables are accessories for the diagnostic equipment (recording/monitoring device) and the EMG needle electrodes, and as such cables are intended to be used by a qualified healthcare professional. This product can be used on the general population at the discretion of a healthcare professional.

Use of product

- TE/D64-551 and TE/D64-851 cables must be used in combination with Technomed Europe branded concentric needle electrode or single fiber needle electrode, and a 240-degree 5 pole DIN compatible recording/monitoring device.
- The TE/D54-451 cable must be used in combination with Technomed Europe branded disposable monopolar needle electrode and a 240-degree 5 pole DIN compatible recording/monitoring device.
- The TE/D34-231 cable must be used in combination with Technomed Europe branded disposable monopolar needle electrode and a DIN 42802 compatible recording/monitoring device.

Cleaning

Reusable EMG electrode cables should be thoroughly cleaned after each use. Immediately after use, wipe the cable using a soft cloth dipped in lukewarm water (at temperatures below 43°C (110°F)) with a mild detergent/enzymatic product to remove foreign material. Excessive pressure and continued flexing of the cable during cleaning or use can stretch the cable jacket and break the internal wires, damaging the cable. After wiping clean, rinse the cable with lukewarm water to prevent coagulation and thus assisting in the removal of protein substances. Carefully inspect the cable to ensure the complete removal of any contamination. Do not immerse or rinse cable connectors in any solution, as it may damage the product.

Note: Avoid contact with strong, aromatic, chlorinated, ketone, ether, or ester solvents. Prolonged immersion (more than 1 hour) in alcohol or mild solvents, detergent solutions or highly alkaline solutions may affect the cable and electrode jacket flexibility.

INSTRUCTIONS FOR USE

REUSABLE EMG ELECTRODE CABLES

Drying

After cleaning, dry the cable with a clean, soft, low-linting cloth to remove any remaining water.

Disinfection and sterilization

Reusable EMG electrode cables are not intended to be disinfected or sterilized, as they are not expected to become infected during their intended use. Use of strong disinfection solutions and any sterilization techniques may damage the product.

Inspection

Visually inspect reusable EMG electrode cables for deterioration or excessive wear prior to each use. If cables or connectors have any fracture, damaged wire insulation, or unnatural angle, discard cables according to your hospital's procedure for medical waste.

Storage and Transport

Protect cables from damage and contamination during storage and transport. When not in use, cables should be loosely coiled and stored at room temperature. Do not wrap cables around the equipment casing to avoid damaging internal wires. When transporting the products outside their main area of use, pack them in a resealable plastic bag to prevent damage or contamination.

Note: All serious incidents associated with the use of reusable EMG electrode cables should be reported to Technomed Europe at qualitytechnomed@welcony.com and to a competent authority of the country where user or patient is established.

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